FOOT MASSAGER SSFM 35 A1

GB

FOOT MASSAGER

Operation and safety notes

HU

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SI

NAPRAVA ZA MASAŽO STOPAL

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

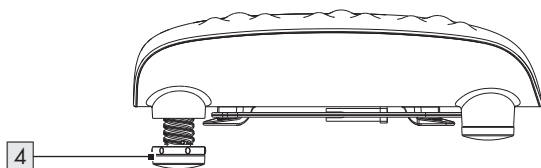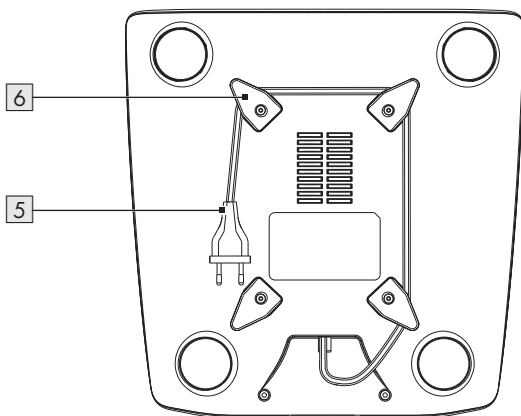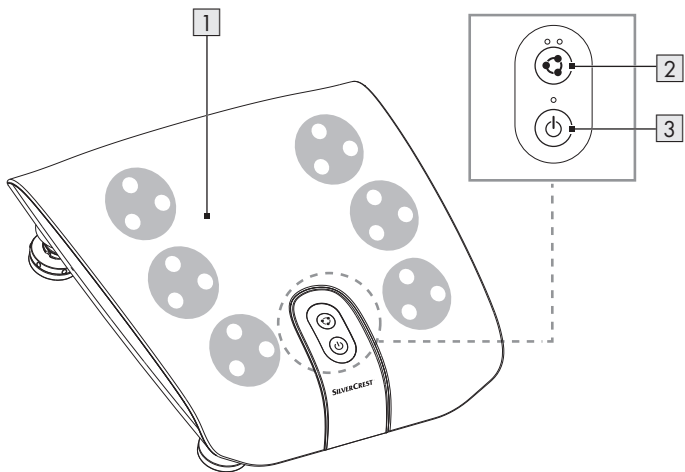
A

| | | |
|--|------|----|
| Warnings and symbols used | Page | 6 |
| Introduction | Page | 6 |
| Intended use | Page | 6 |
| Scope of delivery | Page | 7 |
| Parts description | Page | 7 |
| Technical data | Page | 7 |
| Safety instructions | Page | 7 |
| Before first use | Page | 10 |
| Operation | Page | 10 |
| Setting up the product | Page | 10 |
| Use massage and heat function | Page | 11 |
| Switching off the product | Page | 11 |
| Troubleshooting | Page | 12 |
| Cleaning and care | Page | 12 |
| Storage | Page | 12 |
| Disposal | Page | 12 |
| Warranty | Page | 13 |

Warnings and symbols used

The following warnings are used in this user manual and on the packaging:

| | | | |
|--|--|---|--|
|  | <p>DANGER! This symbol in combination with the signal word “Danger” marks a high-risk hazard that if not prevented could result in death or serious injury.</p> |  | <p>Alternating current/voltage</p> |
| | |  | <p>Hertz (supply frequency)</p> |
| | |  | <p>Watts</p> |
|  | <p>WARNING! This symbol in combination with the signal word “Warning” marks a medium-risk hazard that if not prevented could result in death or serious injury.</p> |  | <p>Danger – risk of electric shock!</p> |
| | |  | <p>Use the product in dry indoor spaces only.</p> |
| | |  | <p>Symbol for a Protection Class II product</p> |
|  | <p>CAUTION! This symbol in combination with the signal word “Caution” marks a low-risk hazard that if not prevented could result in minor or moderate injury.</p> |  | <p>NOTE: This symbol in combination with “Note” provides additional useful information.</p> |
| | |  | <p>CE mark indicates conformity with relevant EU directives applicable for this product.</p> |

FOOT MASSAGER

● Introduction

We congratulate you on the purchase of your new product. You have chosen a high quality product. The instructions for use are part of the product. They contain important information concerning safety, use and disposal. Before using the product, please familiarise yourself with all of the safety information and instructions for use. Only use the product as described and for the specified applications. If you pass the product on to anyone else, please ensure that you also pass on all the documentation with it.

● Intended use

This product can be used for foot massage in the Shiatsu style (Japanese for: shi = finger and atsu = pressure). Shiatsu is a form of body massage developed in Japan about 100 years ago and is based on the ideas of traditional Chinese medicine. It is based on the system of energy channels (meridian system) in the human body.

The product is designed for household use. The product is not suitable for commercial use.

The manufacturer accepts no liability for damages caused by improper use.

● Scope of delivery

After unpacking the product, make sure that the product is in orderly condition. Remove all packaging materials before use.

- 1x Foot massager
- 1x Instruction manual

● Parts description

(Fig. A)

- 1 Massage surface
(with 9 Shiatsu massage heads on both sides)
- 2  (speed/warming button with indicator light)
- 3  (on/off button with indicator light)
- 4 Height-adjustable feet
- 5 Power cord and power plug
- 6 Power cord storage

● Technical data

Input voltage: 220–240 V~, 50–60 Hz
Protection class: II/□
Power consumption: 35 W

Safety instructions

BEFORE USING THE PRODUCT, PLEASE FAMILIARISE YOURSELF WITH ALL OF THE SAFETY INFORMATION AND INSTRUCTIONS FOR USE! WHEN PASSING THIS PRODUCT ON TO OTHERS, PLEASE ALSO INCLUDE ALL THE DOCUMENTS!

In the case of damage resulting from non-compliance with these operating instructions the warranty claim becomes invalid! No liability is accepted for consequential damage! In the case of material damage or personal injury caused by incorrect handling or non-compliance with the safety instructions, no liability is accepted!

Children and persons with disabilities

⚠️ WARNING! RISK OF LOSS OF LIFE OR ACCIDENT TO INFANTS AND CHILDREN!

Never leave children unsupervised with the packaging material. The packaging material represents a danger of suffocation.

Children frequently underestimate the dangers. Always keep children away from the packaging material.

- This product can be used by children aged from 8 years and above and persons with reduced physical, sensory or mental capabilities, or lack of experience and knowledge if they have been given supervision or instruction concerning use of the product in a safe way and understand the hazards involved.

- Children shall not play with the product.
- Cleaning and user maintenance must not be performed by children without supervision.

⚠️ CAUTION! The product has a heated surface. Persons insensitive to heat must be careful when using the product.

Intended use

⚠️ WARNING! Misuse may lead to injury. Use this product solely in accordance with these instructions.

Do not attempt to modify the product in any way.

Electrical safety

⚠️ DANGER! Risk of electric shock! Never attempt to repair the product yourself. In case of malfunction, repairs are to be conducted by qualified personnel only.

⚠ WARNING! Risk of electric shock! Do not immerse the product's electrical components in water or other liquids. Never hold the product under running water.

⚠ WARNING! Risk of electric shock! Never use a damaged product. Disconnect the product from the power supply and contact your retailer if it is damaged.

- Regularly check the power plug and the supply cord for damages. If the supply cord is damaged, it must be replaced by the manufacturer, its service agent or similarly qualified persons in order to avoid a hazard.
- Ensure the rated voltage shown on the rating label corresponds with the voltage of the power supply.
- Do not insert needles or pointed objects into the massage surface.

Use

⚠ WARNING! Risk of injury! Turn off the product and disconnect it from the mains before cleaning and when the product is not in use.

- The product must be used and rested on a flat, stable surface.
- The product is not to be used if it has been dropped, or if there are visible signs of damage.
- Do not operate the product with a damaged power cord or plug, when the product malfunctions or has been damaged in any manner.
- Only use the product in dry indoor rooms.
- Do not use the product with wet feet or while the product is standing on a wet floor. Do not touch the power plug with wet hands.

- Protect the product against heat. Do not place the product close to open flames or heat sources such as stoves or heating appliances.
- Protect the power cord against damages. Ensure that nobody can pull on or trip over it unintentionally.
- Do not pull the power plug out of the power outlet by the power cord. Do not wrap the power cord around the product.
- Protect the product, its power cord and power plug against dust, direct sunlight, dripping and splashing water.
- Store the product in a cool, dry place, protected from moisture and out of the reach of children.

● **Before first use**

- Remove the packing materials.

● **Operation**

⚠ CAUTION! Risk of injury! Do not press any parts of the body, especially fingers, between the rotating massage heads **1** and their holders in the product.

❗ **NOTES:**

- Never stand on the product.
- Do not use the product for more than 15 minutes. Prolonged use may cause over stimulation of muscles, leading to tension.
- Do not use the product on swollen, burnt, inflamed, diseased or injured skin areas.
- Always remove your shoes before use.
- Use the product only with dry and clean feet.
- The heat function can only be switched on, if the massage function is switched on.
- The product switches off automatically after 15 minutes. Let the product cool down before switching it on again.

● **Setting up the product**

- Unroll the power cord **5** from the power cord storage **6**.
- For an optimal massage position, the rear feet **4** can be adjusted. Place the product on a flat surface with the massage surface **1** facing downwards.
- Rotate the feet **4** as follows to adjust the foot height:

| Direction of rotation | Foot height |
|-----------------------|-------------|
| Clockwise | Increasing |
| Counter-clockwise | Decreasing |

- Place the product on a flat surface.
- Slippage of the product is prevented by the rubber feet **4** on the bottom of the product. If necessary, place the product on a mat to protect the floor.
- Check the stability of the product.


● Use message and heat function

| Setting | Massage level | Heat function | 🔄 button 2 |
|---------|---------------|---------------|-------------------|
| 1 | Slow | On | Lights green |
| 2 | Slow | Off | Flashes green |
| 3 | Fast | On | Lights blue |
| 4 | Fast | Off | Flashes blue |


- Connect the power plug **5** to a power outlet.
- Switching on the product:
 - Press the ⏻ button **3**.
 - The indicator light of the ⏻ button **3** lights up red.
 - The massage heads **1** rotate slowly.
 - The heat function is switched on.
 - The left indicator light of the 🔄 button **2** lights up green.
 - The product is set to massage level 1.
- Assume a comfortable sitting posture. Place your feet on the massage heads **1**.
- This product is equipped with 2 adjustable massage levels, each with a switchable heat function (see table).

Change settings: Press the 🔄 button **2** again.

Each time you press the 🔄 button **2**, the setting increases: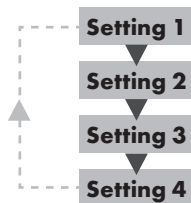

One of the indicator lights of the 🔄 button **2** lights up or flashes, depending on the selected setting:

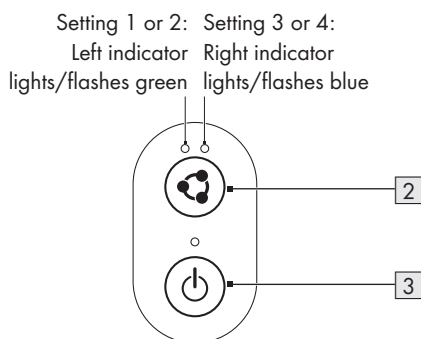● Switching off the product

i NOTE: The product is equipped with a temperature sensor. When the temperature reaches a certain level, the heat function switches off temporarily. The heat function switches on again when the product has cooled.

- Switching off the product:
 - Press the ⏻ button **3**.
 - The massage heads **1** no longer rotate.
 - The indicator lights of the ⏻ **3** and 🔄 buttons **2** go off.
- Switching off the product completely: Unplug the power plug **5** from the power outlet.

● **Troubleshooting**

| Problem | Cause | Solution |
|---------------------------------------|---|---|
| Massage heads 1 rotate slowly. | Pressure on massage heads 1 is too strong. | Remove pressure from massage heads 1 . |
| Massage heads 1 do not move. | Power plug 5 is not connected to a power outlet. | Connect the power plug 5 with a power outlet. Switch on the product. |

● **Cleaning and care**

⚠ DANGER! Risk of electric shock!

Before cleaning or maintaining the product:
Disconnect the power plug **5** from the power outlet.

⚠ WARNING! Risk of electric shock!

Do not immerse the product in water or other liquids. Never hold the product under running water.

- Clean the product with a slightly damp cloth.
- Do not allow water or other liquids to get inside the product.
- Do not use any abrasive, aggressive cleaners to clean the product.
- After cleaning: Allow the product to dry.

● **Storage**

- Roll the power cord **5** onto the power cord storage **6**.
- Store the product in the original packaging when it is not in use.
- Store the product at a dry location out of the reach of children.

● **Disposal**

The packaging is made entirely of recyclable materials, which you may dispose of at local recycling facilities.

Observe the marking of the packaging materials for waste separation, which are marked with abbreviations (a) and numbers (b) with following meaning: 1-7: plastics / 20-22: paper and fibreboard / 80-98: composite materials.

The product and packaging materials are recyclable, dispose of it separately for better waste treatment.

The Triman logo is valid in France only.

Contact your local refuse disposal authority for more details of how to dispose of your worn-out product.

To help protect the environment, please dispose of the product properly when it has reached the end of its useful life and not in the household waste. Information on collection points and their opening hours can be obtained from your local authority.

● **Warranty**

The product has been manufactured to strict quality guidelines and meticulously examined before delivery. In the event of product defects you have legal rights against the retailer of this product. Your legal rights are not limited in any way by our warranty detailed below.

The warranty for this product is 3 years from the date of purchase. The warranty period begins on the date of purchase. Please keep the original sales receipt in a safe location. This document is required as your proof of purchase.

Should this product show any fault in materials or manufacture within 3 years from the date of purchase, we will repair or replace it – at our choice – free of charge to you. This warranty becomes void if the product has been damaged, or used or maintained improperly.

The warranty applies to defects in material or manufacture. This warranty does not cover product parts subject to normal wear, thus possibly considered consumables (e.g. batteries) or for damage to fragile parts, e.g. switches, rechargeable batteries or glass parts.

Warranty claim procedure

To ensure quick processing of your case, please observe the following instructions:

Please have the till receipt and the item number (IAN 394266_2104) available as proof of purchase.

You will find the item number on the rating plate, an engraving, on the front page of the instructions for use (bottom left), or as a sticker on the rear or bottom of the product.

If functional or other defects occur, please contact the service department listed either by telephone or by e-mail.

You can return a defective product to us free of charge to the service address that will be provided to you. Ensure that you enclose the proof of purchase (till receipt) and information about what the defect is and when it occurred.

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OWIM GmbH & Co. KG